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## Jen-LC Flow [ FB ]

EN Flowable restorative composite fiber reinforced

Last reviewed on October 12, 2025

For professional use in dentistry only!

PRODUCED BY: JENDENTAL-UKRAINE LLC

Boryspilska Str. 9, Build. 61, Kyiv, 02099, Ukraine +380 44 33 787 82

www.jendental-ukraine.com

## Jen-LC Flow FB

EN

Jen-LC Flow FB is a light-cured, fiber reinforced, flowable restorative material intended for intraoral use.

### INDICATIONS

- Direct composite restorations: large cavities, including those missing up to three surfaces; deep carious lesions; endodontically treated teeth, including those reinforced with glass-fiber posts; cavities with missing cusps; and cavities after amalgam removal.
- Core build-up under prosthetic restorations

**Note:** Despite its high strength and fracture resistance, the material must always be covered with a universal light-cured composite to ensure wear resistance and esthetics.

### DIRECTIONS FOR USE

**1. Preparation:** Remove the cap, attach the dispensing tip, and check that no material remains at the opening.

**2. Shade selection:** UB (Universal Body) or A2, A3.

For strongly discolored dentin or pigmented areas, apply a thin layer of an opaque flowable composite (e.g., Jen LC-Flow UO or equivalent) on the cavity bottom or at the dentin-enamel junction.

**3. Cavity preparation:** Prepare the cavity according to standard techniques.

In deep cavities, use an insulating liner; if necessary, protect the pulp with calcium hydroxide (e.g., Calci-Jen).

**4. Bonding:** Use a light-cured adhesive system following the manufacturer's instructions (e.g., Jen-Unibond, Jen-Unibond SE).

### 5. Material placement:

Apply the material through the disposable dispensing tip (cannula).

To avoid cross-contamination, each cannula must be used only once.

Condense the material carefully to ensure close adaptation to cavity walls and bottom, leaving 1–2 mm space for the universal composite overlay.

For Class II or large cavities, first rebuild the missing walls with universal composite (approx. 2 mm thick) to stabilize the restoration before placing Jen-LC Flow FB.

Most cavities can be filled in one layer up to 4 mm.

If the cavity depth exceeds 4 mm, apply in several 2–3 mm layers, light-curing each separately.

If the cured material requires contouring, conventional rotary instruments may be used per manufacturer's guidance.

Protect the material from strong light to prevent premature polymerization.

### IMPORTANT!

- The material should be used at room temperature. Cold material is more viscous and may be more difficult to extrude.
- If extrusion is difficult, remove the tip and dispense directly from the syringe.
- After use, always remove the dispensing tip and close the syringe with its cap to prevent polymerization inside the tip due to ambient light exposure.

### 6. Light Curing:

Curing depth	Curing time (LED > 1200 mW/cm <sup>2</sup> )	Curing time (LED/Halogen > 700 mW/cm <sup>2</sup> )
up to 4 mm	20 s	30 – 40 s

Use a polymerization lamp suitable for materials containing camphorquinone, with an emission peak near 465 nm.

Minimum irradiance required: 700 mW/cm<sup>2</sup>.

**Note:** Lower light intensity may cause incomplete curing and reduced mechanical and chemical properties.

**7. Covering Composite:** Apply a universal wear-resistant composite (e.g., Jen-Favorite ZrSi) 1–2 mm occlusally, following the manufacturer's instructions.

Optimal esthetics can be achieved by selecting appropriate shades and translucency (enamel, dentin, or body tones) based on the clinical situation.

Jen-LC Flow FB must always be used in combination with a universal restorative composite and cannot be used alone for replacing occlusal or vestibular enamel.

**Finishing and Polishing:** Finishing and polishing should follow standard protocols recommended by the manufacturer of the universal composite material used for final layer.

### CONTRAINDICATIONS

- Direct pulp capping.
- Known allergy to methacrylate monomers or polymers.
- Use as a final surface layer without an overlying universal composite.

### CLEANING AND DISINFECTION

Reusable components must be cleaned and disinfected using intermediate-level disinfectants. Do not immerse in liquid solutions.

### PRECAUTIONS

1. Wear protective eyewear, clothing, mask, and gloves when handling the material. Provide protective glasses for the patient as well.
2. Follow standard safety procedures for handling powder-containing, silicon-based materials.

### SIDE EFFECTS

The material may cause irritation of eyes, skin, or mucous membranes (see Warnings).

### WARNINGS

Jen-LC Flow FB contains methacrylate resins.

Avoid prolonged or repeated contact of uncured material with skin, oral soft tissues, or eyes.

In case of contact, rinse immediately with soap and water.

If skin rash or allergic reaction occurs, discontinue use and seek medical advice.

### INTERACTIONS WITH OTHER DENTAL MATERIALS

Do not use eugenol-containing materials in combination with Jen-LC Flow FB. Eugenol-based products may inhibit methacrylates polymerization.

### WARRANTY

The manufacturer guarantees product quality.

The warranty does not cover defects resulting from misuse, violation of instructions, or improper storage.

The user is responsible for determining product suitability for intended use.

In the event of proven manufacturing defects, the product will be replaced.

### MANUFACTURER'S LIABILITY

The manufacturer's liability is limited to cases defined by the applicable laws of the respective country.

### STORAGE AND SHELF LIFE

- Store at 4–25 °C.
- Always close the cap tightly after use.
- Shelf life: 3 years.
- Do not freeze!

### DISPOSAL

Dispose of the medical device according to local, regional, national, or international regulations.

### PACKAGING

Jen-LC Flow FB is supplied in syringes of 2 g or 5 g with a set of dispensing tips. Shades: A2, A3, UB (Universal Body).