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Lubrelief-Paro

UA Lubricating gel for
instrumental tartar removal

For professional use in dentistry only!

PRODUCED BY: JENDENTAL-UKRAINE LLC

Boryspilska Str. 9, Build. 61, Kyiv, 02099, Ukraine +380 44 33 787 82

www.jendental-ukraine.com

Lubrelief-Paro

Lubricating gel for instrumental tartar removal

Lubrelief-Paro is used to lubricate instruments during mechanical or ultrasonic tartar removal procedures. The material contains cetyltrimethylammonium bromide (or CTAB), a cationic surfactant with antiseptic properties. The presence of this component in the material provides the following properties:

1. Reduction of surface tension

CTAB molecules have a hydrophobic "tail" that interacts with fats/proteins. A hydrophilic "head" with a positive charge that is attracted to water and negatively charged surfaces (bacteria, enamel). This reduces the surface tension of the fluid in the surgical field, allowing it to:

- Penetrate deeper into the pores of tartar.
- Weaken its adhesion to enamel.

2. Destruction of biofilm

• CTAB disrupts the lipid and protein structure of the biofilm matrix. This facilitates its separation during ultrasonic vibration.

3. The antimicrobial effect CTAB (like all soaps and surfactants) is able to inhibit the growth of microorganisms and their colonization of surfaces (including the formation of colonies in liquids and gels), especially when interacting with gram-positive bacteria. This helps maintain the stability of the gel itself, increases its shelf life. Limits the likelihood of the spread of pathogenic microorganisms along with the remains of dental deposits

4. Detergent action

CTAB acts as a detergent, emulsifying and dispersing fatty components:

- Cleans organic contaminants (proteins, lipids).
- Facilitates mechanical removal of tartar.
- Improves the penetration of ultrasound and irrigation solution into microcracks and the subgingival area.

The material has low toxicity, does not cause allergic reactions or chemical burns to the mucous membrane. It is easy to apply in hard-to-reach areas. Lubrelief-Paro is economical, easy and simple to use, and has a pleasant taste. It has an additional antibacterial effect and reduces the sensitivity of soft tissues and teeth in the cervical area during mechanical or ultrasonic tartar removal procedures.

INDICATIONS FOR USE

Lubrication of instruments to facilitate work during instrumental or ultrasonic removal of tartar. Reduction of sensitivity of soft and hard tooth tissues in the cervical area during scaling. Used as a lubricant during cofferdam application. Facilitates work and reduces discomfort during procedures to remove excess cement, when installing crowns or bridges, curettage, or during the removal of dental deposits.

COMPOSITION

Contains: cetyltrimethylammonium bromide, polyethylene glycol, propylene glycol, benzocaine, dye solution (water, ethyl alcohol, dye), flavoring, thickener.

METHOD OF APPLICATION

Use a saliva ejector to remove excess saliva from the patient's mouth.

When applying the gel from a syringe: remove the protective cap and squeeze out the required amount of Lubrelief-Paro. If using gel from a jar, apply it with a tool or a clean cotton swab. Do not use excessive amounts of gel. If necessary, the product can be reapplied.

FLAVORINGS: Cherry / Strawberry.

PRECAUTIONS AND WARNINGS

Do not use Lubrelief-Paro on patients who are allergic or hypersensitive to cetyltrimethylammonium bromide, benzocaine, or other components of the gel. Do not use Lubrelief-Paro on pregnant women and children under 6 years old. If the patient develops a rash or other signs of an allergic reaction, discontinue use and seek medical advice. Do not use for injections.

POSSIBLE COMPLICATIONS

Allergic reactions, irritation, and redness of the mucous membrane.

WARRANTY OBLIGATIONS

The manufacturer guarantees the quality of its products. The warranty does not cover defects resulting from failure to follow the instructions for use of the materials, violation of storage conditions, and other cases not provided for by the functional purpose of the material.

The consumer is responsible for determining the suitability of this product for use in accordance with his (the consumer's) tasks. Condition for warranty coverage: non-compliance with the product with the characteristics declared by the manufacturer. If a defect in the material is detected during the warranty period, the manufacturer will replace the product.

LIMITATION OF MANUFACTURER'S LIABILITY

The manufacturer's liability extends only to cases expressly provided for by the current legislation of the country.

STORAGE AND SHELF LIFE

Store at a temperature of 4–25 °C, close the cap tightly after use.
Shelf life: 3 years. **DO NOT FREEZE!**

DISPOSAL

Dispose of the medical device in accordance with national/international legal requirements.

PACKAGING

Lubrelief-Paro in 30 g/g jars; 20 g/g jars.

Lubrelief-Paro in 5 ml/ml syringes

Last revised June 24, 2025.