•• INDICATION

- Stopping bleeding and reducing inflammation in gum retraction, performing fingerprint procedures, restoring class V, and more.
- Reduction of excretion of sucral fluid during fixation of indirect adhesive restorations.
- The drug is intended for local application and can be used both with impregnated, and with impenetrable threads.
- It does not stain the sprained areas and tooth fabrics.

•• COMPOSITION

- Aluminum Sulfate U.S.P. – 25.0%
- 8-Hydroxyquinoline – 0.1%
- Colored Gel – ad 100%

•• INDICATIONS

For the purpose of aiding in gingival retraction and to control bleeding and reduce mucous membrane inflammation associated with dental impression procedures and Class V restorations or other tissue management problems in other phases of dentistry. A more recent use is to assure sulcular fluid control during luting of indirect bonded restorations. This product is for topical use only. Alu-Jen Gel is compatible with any type of impregnated or plain cord. Free from coloring the area prepared.

•• DIRECTIONS FOR USE

Attach delivery tip to Alu-Jen Gel syringe. Apply a thin bead of gel directly into the sulcus area. Select cord and pack into gel. Wait 2 – 6 minutes. If still bleeding, rinse with air/ water spray and reapply gel. Remove cord. Rinse and dry. Take impression as usual.

•• SPECIAL RECOMMENDATIONS

The use of double cord pack technique is recommended. Place a smaller cord first, followed by a larger size. The small cord can be left in while the impression is taken and then removed.

•• CONTRAINDICATIONS

- None.

•• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty.

The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

•• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

•• STORAGE

- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
- Shelf life – 2 years.
- DO NOT FREEZE!

•• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

•• PACKING

- 5 ml in syringe delivery tips.

JenD[®]

For professional use in dentistry only!

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Gel lubricant for lubrication of the tool during instrumental retraction of the gums when removing silicone prints

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