•• INDICATIONS

- For softening and desobturation of the root canals sealed with the phenol, formalin, and formaldehyde resins.

•• DIRECTIONS FOR USE

Desobturation of the canals is performed by combined chemical action of Jen-Desobturat with mechanical reaming:

- Clean out the pulp chamber and canal orifice.
- Place a drop of Jen-Desobturat into the canal orifice.
- Desobturate the canal using a broach. After extraction out of the canal clean and put the tip of instrument into the Jen-Desobturat solution. Repeat the procedure step by step till the apex is reached.

•• PRECAUTION

Avoid contact with eyes and minimize contact with oral soft tissue. If accidental contact with eye occurs, flush immediately with large amount of water.

•• COMPOSITION

- DMFA – 98.6 g
- Excipient Q.S. – 100.0 g

•• SUPPLEMENTARY INDICATIONS

Dry thoroughly the canal treated with Jen-Desobturat before filling by formalin/formaldehyde cement.

•• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty.

The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

•• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

•• STORAGE

- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
- Shelf life – 2 years.
- DO NOT FREEZE!

•• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

•• PACKING

- Bottle 9 ml.

JenD[®]

For professional use in dentistry only!

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